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device carries out a self test. During this test, internal functional faults, or connection faults will be determined and will appear in form of an error code on the display. The alarm relay will be switched during this test.

Whilst the test button "T" is pressed, all display elements available for this device are shown.

Functional faults

In case of a malfunction, the relay K1 (11, 12, 14) and all of the three LEDs flash. An error code appears on the display.

E01 = PE connection fault, no low-resistance connection between E and KE.

E02 = System connection fault, no low-resistance connection between L1 and L2.

E03 = Measuring current transformer interruption

E04 = Short-circuit measuring current transformer

E05...Exx = Internal device error, contact the Bender service.

Password protection

If password protection has been activated (on), settings can only be made subject to the correct password being entered (0...999).

Factory setting FAC

After activating the factory setting, all settings previously changed are reset to delivery status.

Monitoring the IT system transformer

The device monitors the temperature by evaluating the resistance value of a PTC. When a response value of 4 kΩ is reached, an alarm will be activated indicating overtemperature, the display shows > °C. Temperature values will not be indicated.

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In order to avoid transformer overload, load current monitoring should be combined with temperature monitoring.

For details refer to the wiring diagram!

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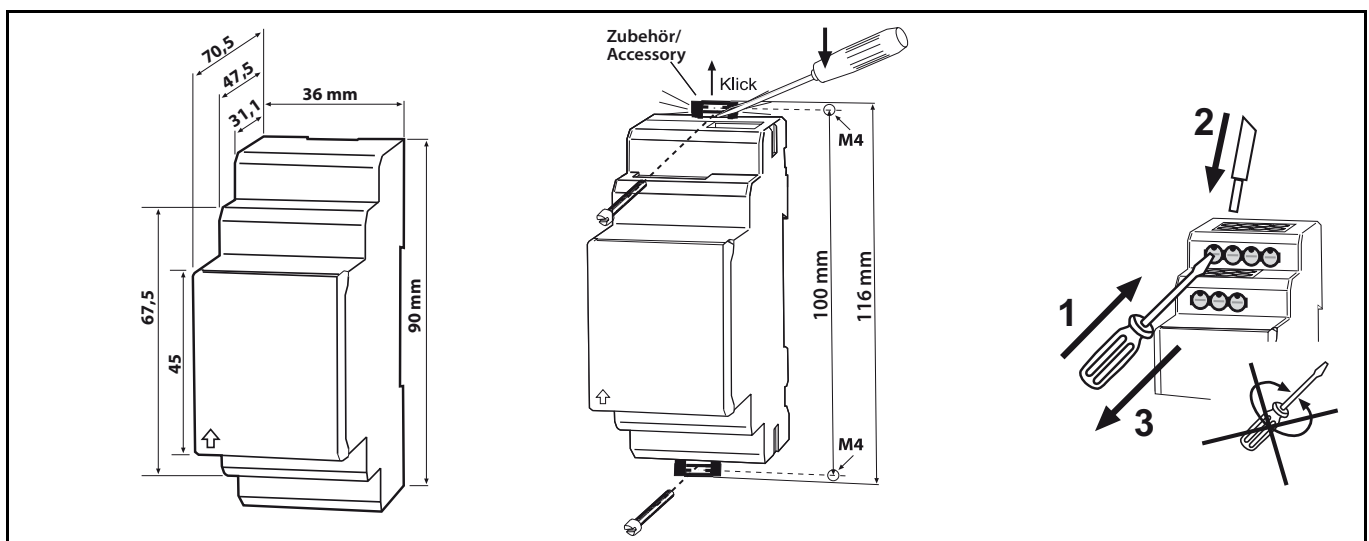
Installation and connection

Ensure safe isolation from supply in the installation area.

Observe the installation rules for live working.

- The isoMED427P is suitable for DIN rail mounting acc. to IEC 60715 or for screw mounting, as described below.

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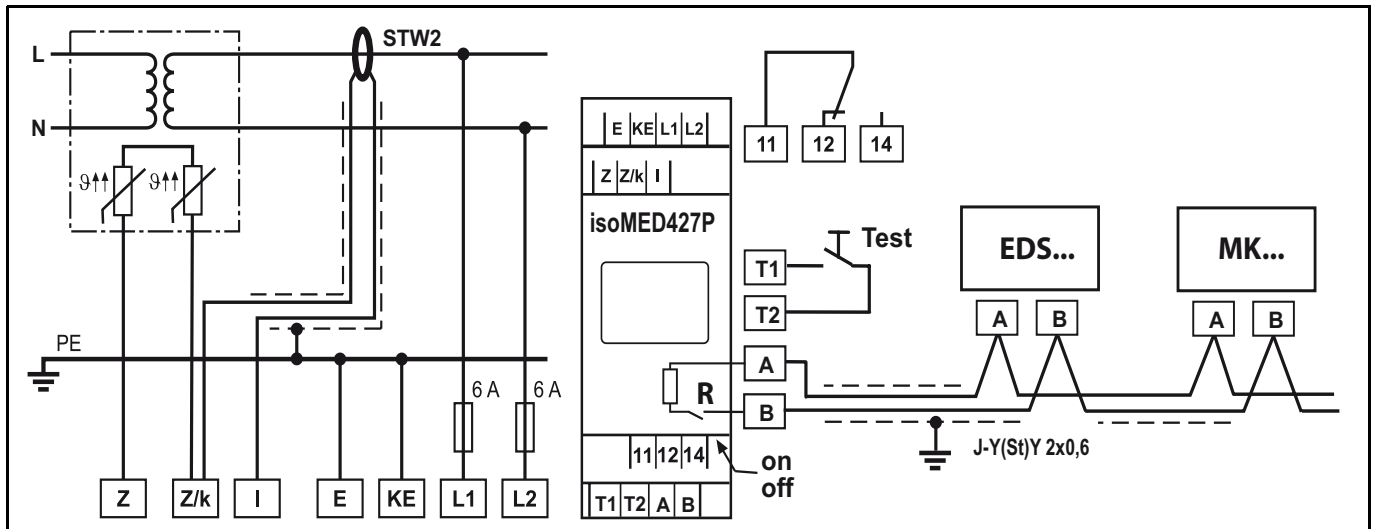
Wiring diagram

If the terminals L1, L2 of the device are connected to an IT system that is energised for operational reasons, the terminals E and KE must not be disconnected from the protective conductor (PE).

Connect the device according the wiring diagram.
The leads to KE and E have to be connected separately!
The BMS bus must be terminated at both ends!

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Terminal	Connections
E, KE	Connect the leads E and KE separately to PE
L1, L2	Connection to the IT system being monitored. Supply voltage (see nameplate) 6 A fuse recommended
Z, Z/k	Connection to the temperature sensor (PTC)
Z/k, I	Connection to the measuring current transformer (STW2)
T1, T2	Connection for external test button
A, B	RS-485 interface, Terminate the connection with switch R (on, off) if the device is connected at the end of the bus.
11, 12, 14	Alarm relay K1

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Commissioning

Prior to commissioning, check proper connection of the A-ISOMETER®.

It is recommended to carry out a functional test using a genuine earth fault, e.g. via a suitable resistance!

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
Display in standard mode

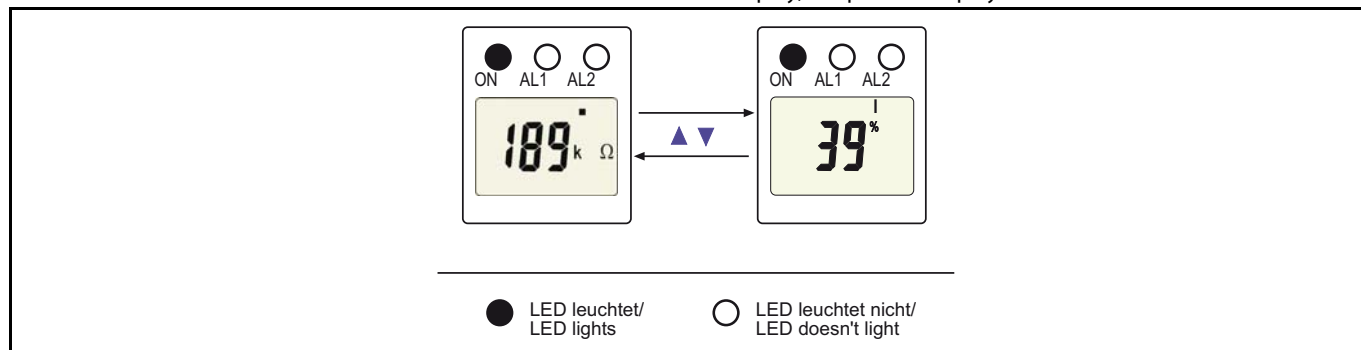If there is no alarm pending, the ON LED lights and the currently measured values are shown on the display. The Up and Down buttons are used to switch over between the indication of the insulation resistance value and the currently measured load current in percentages. If the Enter button is pressed after changing the display, the present display remains.

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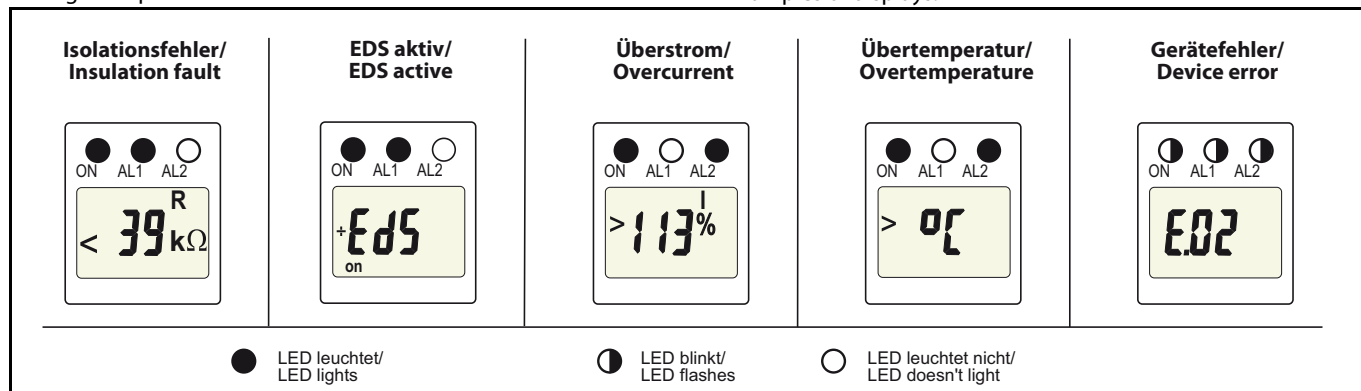
If different alarms occur simultaneously, the display indication changes at 3-second intervals.

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Factory setting

Response value R_{an} : 50 k Ω (< R)
 Response value I_{alarm} : 7 A (> I)
 Response value °C: 4 k Ω (fixed value, for PTC measurement)
 Operating principle K1: N/C (n.c.) operation
 BMS address: 3
 Automatic insulation fault location: off, deactivated
 Password: 0, disabled
 CT monitoring: on, activated
 Termination: off, activated (120 Ω)

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Parameter settings

An example is given below on how to change the alarm response value R_{an} (< R). Proceed as follows:

1. Keep the "MENU / Enter" button pressed for 2 seconds. The flashing short symbol < R appears on the display.
2. Confirm with Enter. A flashing display shows the currently set response value in $k\Omega$.
3. Use the „Up“ or „Down“ button to set the appropriate value. Confirm with „Enter“. The flashing short symbol < R appears on the display.
4. You can exit the menu by:
 - pressing the „Enter“ button for 2 s
 - or selecting the menu item ESC and confirming with „Enter“.

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The areas of the display which can be configured flash! In the figures below, the segments where device settings can be carried out are highlighted by an oval.

50

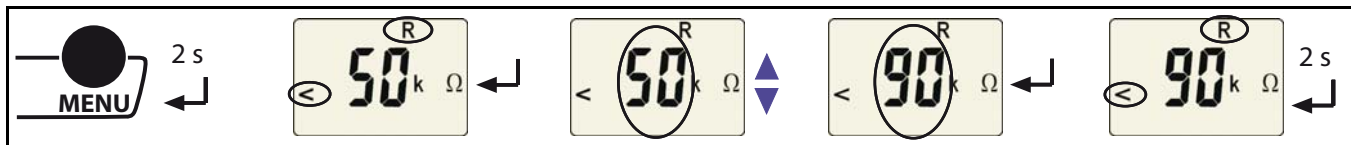
The menu mode can be accessed by pressing the „MENU“ button for 2 seconds.

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
Setting the response value R_{an} (< R)

Use this menu to set the limit of the insulation value at which an alarm is to be signalled.

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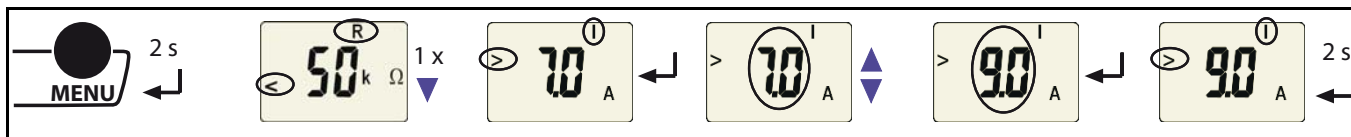Setting of the response value (A) for load current monitoring

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

Selecting the operating principle of the alarm relay

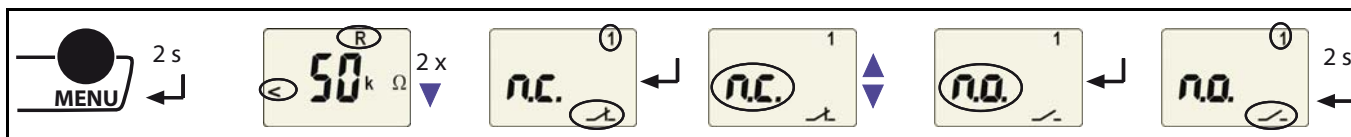Use this menu to set the operating principle of the alarm relay K1 (1) to N/O operation (n.o.) or N/C operation (n.c.):

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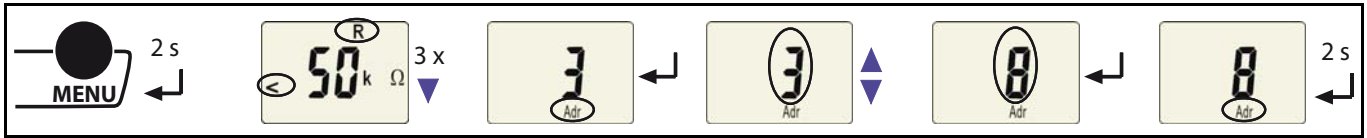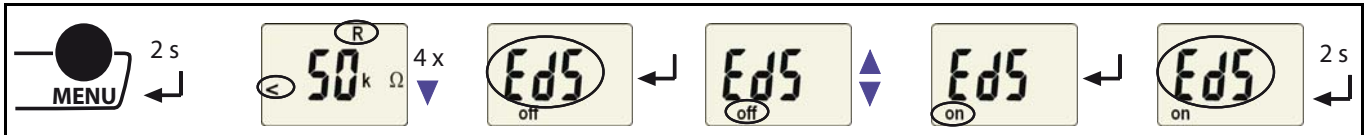
Activating automatic insulation fault location

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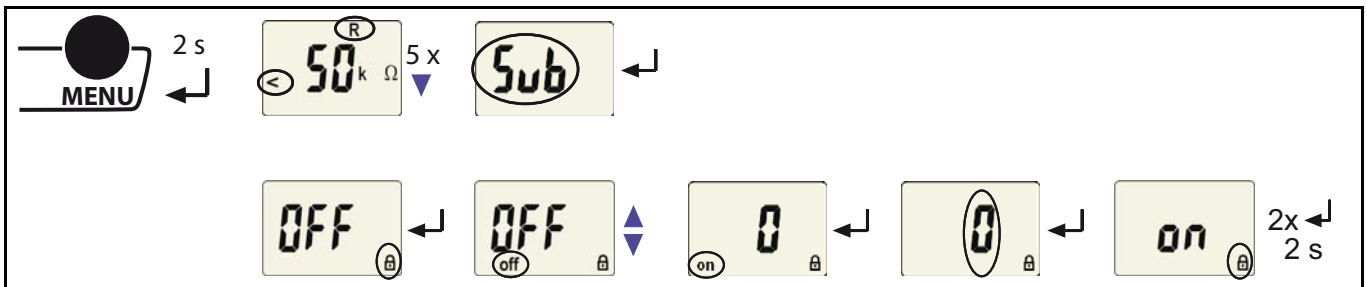
Password protection

This menu can be used to activate password protection, to modify the password or to deactivate password protection. The password can only be changed when the password protection has been activated before.


a) Passwort-Schutz aktivieren

a) Activating the password protection


b) Passwort ändern

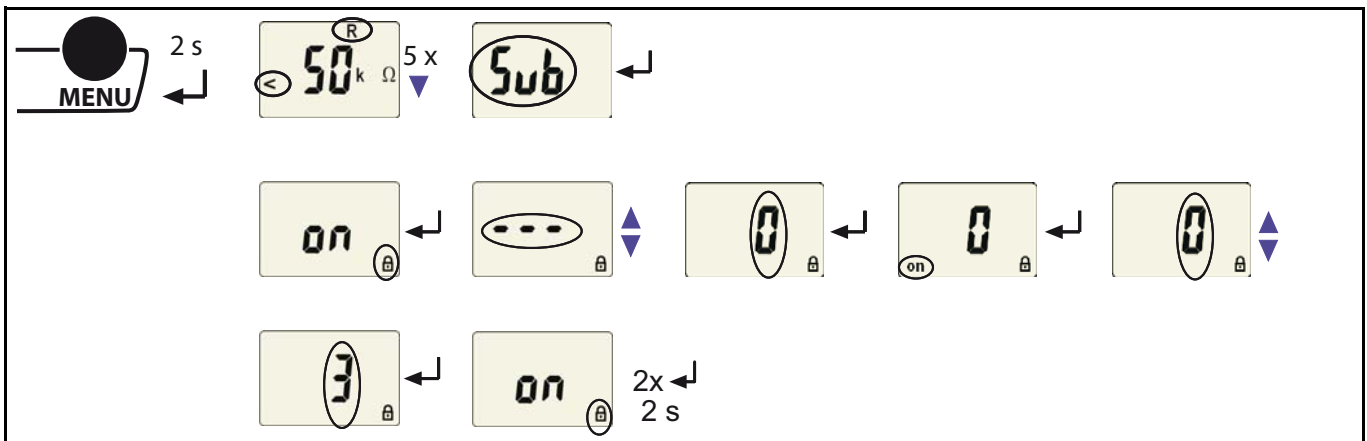b) Changing the password


c) Passwort-Schutz deaktivieren

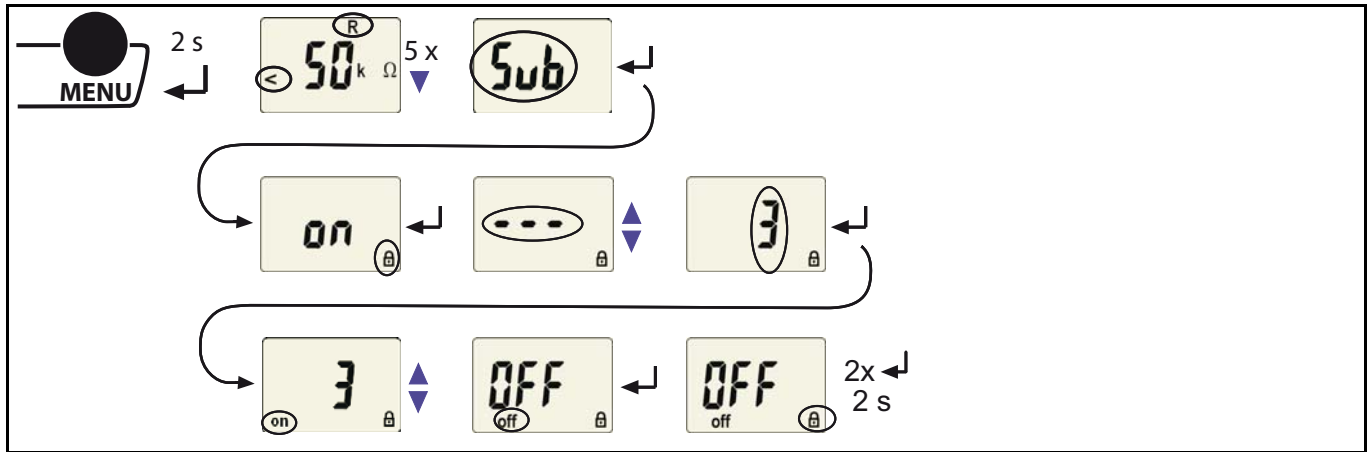c) Deactivating the password protection

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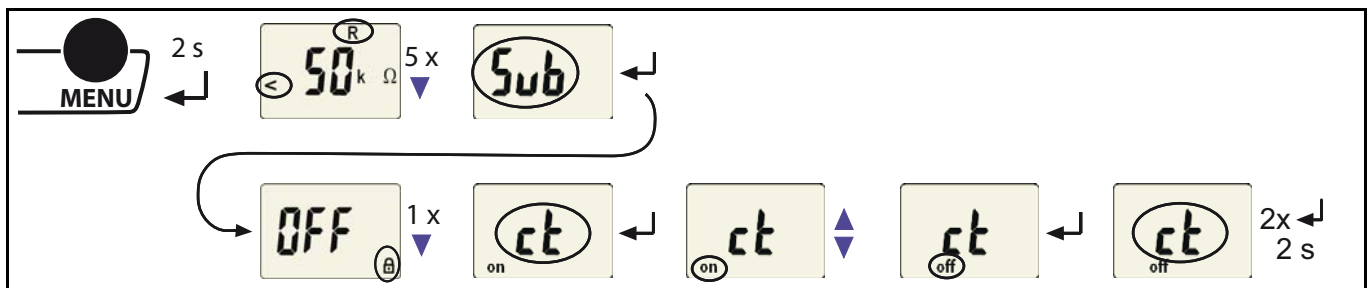
Deactivating the CT monitoring function

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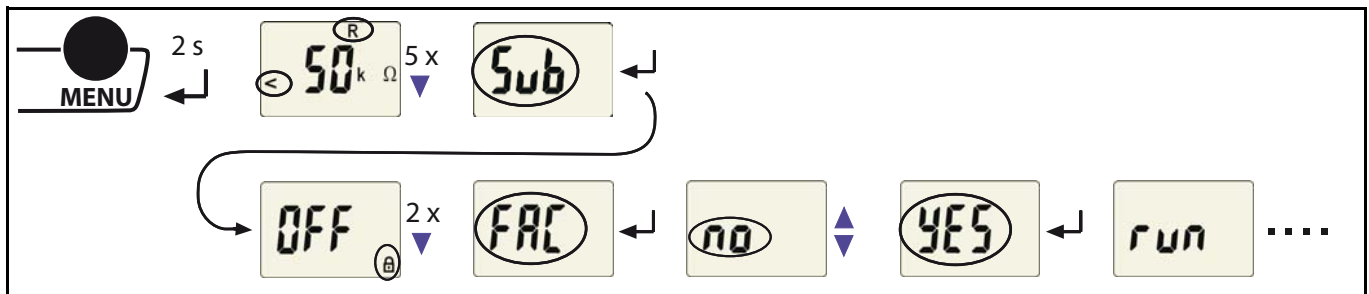Resetting the device to its factory settings

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
Querying device information

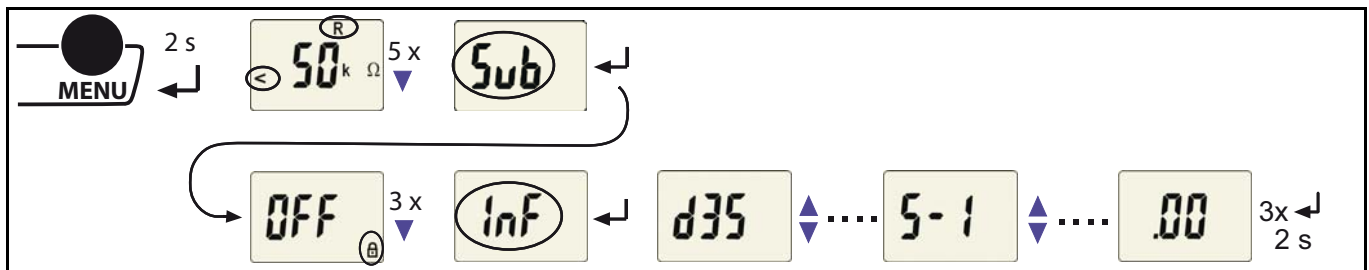Use this menu to query the software version (1.xx). After activating this function, data will be displayed as a scrolling text. Once one pass is completed you can select individual data sections using the Up/Down buttons.

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Technical data

Insulation coordination acc. to IEC 60664-1 / IEC 60664-3

Rated insulation voltage.....	250 V
Rated impulse voltage/pollution degree	2.5 kV / III
Protective separation (reinforced insulation) between:	
..... (L1, L2, E, KE, T1, T2, A, B, Z, Z/k, I) - (11, 12, 14)	
Voltage test according to IEC 61010-1	2.21 kV

Supply voltage

Supply voltage U_S	= U_n
Power consumption	≤ 4 VA

IT system being monitored acc. IEC 60364-7-710:2002-11

Nominal system voltage U_n	AC 70...264 V
Rated frequency f_n	47...63 Hz

Insulation monitoring acc. to IEC 61557-8: 2007-01

Response value R_{an}	50...500 k Ω (50 k Ω)*
Relative uncertainty.....	±10 %
Hysteresis.....	25 %
Response time t_{an} at $R_F = 0.5 \times R_{an}$ and $C_e = 0.5 \mu F$	≤ 5 s
Permissible system leakage capacitance C_e	5 μF

Measuring circuit

Measuring voltage U_m	±12 V
Measuring current I_m (at $R_F = 0 \Omega$)	≤ 50 μA
Internal DC resistance R_i	≥ 240 k Ω
Impedance Z_i , at 50 Hz	≥ 200 k Ω
Permissible extraneous DC voltage U_{fg}	≤ DC 300 V

Locating current injector acc. to IEC 61557-9: 2009

Locating current	≤ 1 mA
Test cycle/idle time	2 s / 4 s

Load current monitoring

Response value, adjustable	5...50 A (7 A)*
Relative uncertainty.....	± 5 %
Hysteresis.....	4 %
Setting value load current measurement:	
Transformer	3150 VA 4000 VA 5000 VA 6300 VA 8000 VA 10000 VA
$I_{alarm 1\sim}$	14A 18A 22A 28A 35A 45 A

Temperature monitoring

Response value (fixed value)	4 k Ω
Release value (fixed value).....	1.6 k Ω
PTC resistors acc. to DIN 44081	max. 6 in series
Relative uncertainty.....	± 10 %

Displays, memory

LC display.....	multifunktional, not illuminated
Measured value insulation resistance	10 k Ω ...1 M Ω
Operating error	± 10 %, ± 2 k Ω
Measured value load current (as % of the set response value)	10 %...199 %
Operating error	± 5 %, ± 0.2 A
Password.....	on, off / 0...999 (off, 0)*

Interface

Interface/protocol.....	RS-485 / BMS
Baud rate	9.6 kbit / s
Cable length.....	0...1200 m
Recommended cable (shielded, shield connected to PE at one end)	at least J-Y(St)Y 2 x 0.6
Terminating resistor	120 Ω (0.25 W), internal, switchable
Device address, BMS bus.....	2...90 (3)*

Interfaces for measuring current transformer STW2 and temperature sensor

Cable lengths:	
Single wire > 0.5 mm ²	≤ 1 m
Single wire, twisted > 0.5 mm ²	≤ 10 m

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Twisted pair, shielded > 0.5 mm² ≤ 40 m
 Recommended cable min. J-Y(St)Y 2x0.6; Shield on one side connected to PE

Switching elements

Number 1 changeover contact
 Operating principle N/C operation / N/O operation (N/C operation)*
 Electrical endurance 10.000 cycles
 Contact data acc. to IEC 60947-5-1
 Utilisation category AC-13 AC-14 DC-12 DC-12 DC-12
 Rated operational voltage 230V 230V 24V 110V 220 V
 Rated operational current 5 A 3 A 1 A 0.2 A 0.1 A
 Minimum contact load 1 mA at AC / DC 10 V

Environment / EMC

EMC IEC 61326-2-4
 Operating temperature -25 °C ... +55 °C
 Classification of climatic conditions acc. to IEC 60721:
 Stationary use (IEC 60721-3-3) 3K5 (except condensation and formation of ice)
 Transport (IEC 60721-3-2) 2K3 (except condensation and formation of ice)
 Long-term storage (IEC 60721-3-1) 1K4 (except condensation and formation of ice)
 Classification of mechanical conditions acc. to IEC 60721:
 Stationary use (IEC 60721-3-3) 3M4
 Transportation (IEC 60721-3-2) 2M2
 Long-time storage (IEC 60721-3-1) 1M3

Connection

Connection type **push-wire terminals**
 Connection properties:
 Rigid 0.2...2.5 mm² (AWG 24...14)
 Flexible without ferrules 0.2...2.5 mm² (AWG 24...14)
 Flexible with ferrules 0.2...1.5 mm² (AWG 24...16)
 Stripped length 10 mm
 Opening force 50 N
 Test opening, diameter 2.1 mm

Other

Operating mode continuous operation
 Position of normal use any
 Degree of protection, internal components (DIN EN 60529) IP30
 Degree of protection, terminals (DIN EN 60529) IP20
 Enclosure material polycarbonate
 Flammability class UL94V-0
 Screw mounting 2 x M4
 DIN rail mounting acc. to IEC 60715
 Software version D288 V1.0x
 Weight 150 g
 () * = factory setting

Ordering information

Typ	Supply voltage $U_s = U_n^*$	Art. No.
isoMED427P-2	AC 70...264 V, 42...460 Hz	B 7207 5301

*Absolute values of the voltage range

STW2 Measuring current transformer	B 942 709
ES0107 Temperature sensing element, PTC	B 924 186
XM420 Mounting frame	B 990 994

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BENDER Group

All rights reserved.
Reprinting and duplicating
only with permission of the publisher.
Subject to change!
© Dipl.-Ing. W. Bender GmbH & Co. KG

Dipl.-Ing. W. Bender GmbH & Co. KG
Londorfer Str. 65 • 35305 Grünberg • Germany
Postfach 1161 • 35301 Grünberg • Germany

Tel.: +49 6401 807-0
Fax: +49 6401 807-259

E-Mail: info@bender-de.com
Web: <http://www.bender-de.com>